User Manual

Compass[®] 4 Hardware 4-Point Hardware

permobil

Contact Information

 The Comfort Companies, LLC
1960 South Calhoun Road
New Berlin, Wisconsin 53151 USA

Phone:
U.S.A.: +1 800-736-0925
Outside the USA: +1 618-277-9150
Fax: +1 406-522-8563

Customer Support:
orders.comfort@permobil.com
permobil.com

WARNING

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in serious injury.

CAUTION

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in minor to moderate injury or damage to the equipment or other property.

Incident Reporting

In the event of a serious incident or injury, contact Customer Support.

Contents

Intended Use		2	
Important Safety Information		3-4	
Parts Detail and Specifications		5	
Compass 4 Hardware		6-8	
Alternate Configurations		9-10	
4-Point Hardware		11	
Troubleshooting		12	
Cleaning and Disinfecting			
Maintenance		13	
Storage, Transport, Disposal and Recycling			
Warranty			
Español	14	Norsk	105
Français	27	Suomi	118
Deutsch	40	Português	131
Italiano	53	hrvatski	144
Nederlands	66	Ελληνικά	157
Dansk	79	Slovenščina	170
Svenska	92		

Symbols Glossary

 Manufacturer	 Model number	 Consult instructions for use
 Size (width x length x depth)	 Unique Device Identifier	 Authorized representative in the European Community
 Medical Device	 Quantity	 Authorized representative in Switzerland

Please Note

Supplier: These instructions must be given to the user of this product.
Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.

Products may be covered by one or more U.S. and foreign patents and trademarks, including ACTA-BACK®, ACTA-EMBRACE®, ACTA-Relief, and Compass®.

Product information is changed as needed; current product information is available at permobil.com.

The Comfort Companies, LLC is a part of Permobil.

© 2022, 2023 Permobil

REV 2023-10-23

The following products are featured in this manual:

Compass 4 Hardware (Quick Release)
4-Point Hardware (45-Degree / 90-Degree)

Intended Use

The Compass 4 Hardware is attachment hardware for Comfort Company Back Supports and is intended to provide height, depth, and recline angle adjustments for users in manual wheelchairs. Available options include Quick-Release and 4-point Hardware. The Compass 4 Hardware has a weight limit of up to 400lbs (204kg).

Compatibility:

Compass 4 Hardware is compatible with:

- ACTA-BACK Back Support
- ACTA-EMBRACE Back Support
- ACTA-Relief Back Support

45-Degree 4-Point Hardware is compatible with:

- ACTA-BACK Back Support
- ACTA-Relief Back Support

90-Degree 4-Point Hardware is compatible with:

- ACTA-EMBRACE Back Support

Not available in all markets:

- ACTA-BACK Back Support
- ACTA-EMBRACE Back Support
- 45-Degree 4-Point Hardware
- 90-Degree 4-Point Hardware

24-inch wide back supports require 4-Point Hardware in addition to Compass 4 Hardware.

Compass 4 Hardware is compatible with circular wheelchair canes with a diameter of 3/4 in. (19mm), 7/8 in. (22mm), 1 in. (25mm) and 1 1/8 in. (29mm). The Compass 4 cane clamps are not compatible with other wheelchair canes.

A clinician who is experienced in seating and positioning should determine whether the product is appropriate for the individual's particular seating needs. The product's intended use statement alone should not be relied upon to make that determination.

The medical products featured in this document are intended to be only one part of an overall care regimen that includes all seating and mobility equipment and therapeutic interventions. A clinician who is experienced in seating and positioning should identify this care regimen after evaluating the individual's physical needs and overall medical condition.

A clinician should also evaluate for visual, reading, and cognitive impairment to determine the need for caregiver assistance or the need for other assistive technology, such as large-print instructions, to ensure proper product use.

Important Safety Information

WARNINGS:

- DO NOT use the product without first reading and understanding all of the provided instructions. If the user is unable to perform any tasks in this manual, seek assistance by contacting a clinician, equipment provider, distributor, or Customer Support.
- Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- The product should be installed by the equipment provider or clinician.
- The product must be correctly sized to the wheelchair.
- Skin/soft tissue breakdown can occur due to a number of factors, which vary by individual. Check skin frequently, at least once a day. Redness, bruising, or darker areas (when compared to normal skin) may indicate superficial or deep tissue injury and needs to be addressed. If there is any discoloration to skin/soft tissue, STOP USE immediately. If the discoloration does not disappear within 30 minutes after disuse, immediately consult a healthcare professional.
- DO NOT modify the product or use non-compatible products, materials, accessories or components, which may result in a non-secure installation of the back support, bodily injury, and product damage or failure.
- As with any addition to a wheelchair, the product may affect the wheelchair's stability. Before using the product, consult with a clinician or the equipment provider to assess the need for additional safety features for the wheelchair, such as anti-tip bars or other available options, to ensure stability during use.
- DO NOT lean over the top of the back support. Doing so may cause a change in the center of gravity, and the wheelchair could tip over.
- Use caution when installing, adjusting or removing hardware to avoid pinching or trapping fingers in openings.
- Before using the first time, and then frequently: confirm the tightness of attaching hardware; re-tighten loose components, following the instructions in this manual; check for wear and breakage. DO NOT continue to use if any part cannot be tightened or is broken. Periodically, the product should be thoroughly inspected by the equipment provider.
- DO NOT lift or push a wheelchair using any part of the back support because it may become detached from the wheelchair.

WARNINGS:

- Metal components may become hot if exposed to high temperatures or cold if exposed to low temperatures. The product's surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the product will contact unprotected skin.
- DO NOT expose the product to high heat, open flames or hot ashes. Testing or certification claims, including for flammability, may no longer apply to this device when it is combined with other products or materials. Check testing and certification claims for all products used in combination with this device.
- Check to make sure that the product does not interfere with any other wheelchair components. Interference could jeopardize effectiveness and user safety and cause damage.
- Keep small hardware components out of the reach of children and any individuals who may have a tendency to place inedible objects in their mouths. Swallowing or inhalation may lead to serious injury or death. IMMEDIATELY seek emergency medical assistance. 
- IMPORTANT! TIGHTEN SECURELY BEFORE USE: For all instructions in this manual, all fasteners should be tightened securely, as instructed. If a torque wrench is available, tighten to 9.6 Nm (85 lb-in).

CAUTIONS:

- Protect the product from blunt force shocks that may cause damage and/or breakage.
- The components of the product are not intended to be used separately from the compatible back supports or in conjunction with other non-compatible wheelchair back supports.
- The back support may need to be removed before folding the wheelchair.
- Use caution when installing, adjusting or removing hardware to avoid pinching or trapping fingers in openings.

Important Safety Information, continued

WARNINGS - Motor Vehicle Transportation:

- Failure to pay attention to these warnings could result in severe injury to the individual in the wheelchair or to others.
- Whenever possible, transfer out of the wheelchair installed with the back support and into a manufacturer-installed vehicle seat, and use the vehicle's crash-tested occupant restraint system.
- The back support must be correctly and securely installed, used, and maintained according to the instructions and warnings provided by the wheelchair and hardware manufacturers.
- Install the back support on a wheelchair so the individual is seated in an upright position and the wheelchair is facing forward during transportation in a motor vehicle. The back support has only been crash tested on a wheelchair in the forward-facing position per the test information provided in this operation manual.
- The back support must only be used with wheelchair frames that are equipped with securement points specified in ANSI/RESNA WC-4:2017, Section 19 (WC19) or ISO 7176-19, and that have been successfully crash tested using the methods described in Annex A of WC19 or ISO 7176-19 (respectively).
- The wheelchair with the back support must be used with an effective wheelchair securement system and a properly positioned, crash-tested pelvic and shoulder-belt restraint, or Wheelchair Tiedown and Occupant Restraint System (WTORS), following the manufacturer's instructions.
- The top of the back support should be positioned as close as possible to the top of the individual's shoulder to provide good support in the event of a crash. The distance between the top of the individual's shoulder and the top of the wheelchair back should not exceed 6½ in. (16.5 cm).
- ALWAYS safely store the back support during transport to avoid damage.
- If the product has been involved in an accident during transport, see the "Maintenance" section in this manual.
- For more information about safely transporting wheelchair users, refer to: <https://wc-transportation-safety.umtri.umich.edu/ride-safe-brochure/>.
- Back support-hardware combinations have been dynamically tested by a third-party testing facility for use in a motor vehicle and meets all applicable criteria for wheelchair seating systems proposed in Section 5.1 of ANSI/RESNA WC-4:2012 Section 20 "Wheelchair Seating Systems for Use in Motor Vehicles".
- Test results apply to seating systems intended for use by adults and children with a total body mass greater than 23 kg (51 lb, or approximately 6 years of age) who are transported facing forward in their wheelchairs in all types of motor vehicles.
- To determine whether motor vehicle third-party testing has been performed with a specific back support-hardware and other accessory combination, contact the equipment manufacturer.

Parts Detail and Specifications

Compass 4 Hardware includes: hardware assembly (x2); 5mm hex key; 3mm hex key; 2mm hex key; operation manual

4-Point Hardware includes: hardware assembly (x2) (45-degree or 90-degree); additional metal straps for mounting larger canes; 1/8 in. hex key; 4mm hex key


⚠️ ⓘ IMPORTANT! This manual includes instructions only for the mounting hardware for back supports. Make sure to also read and follow the instructions provided with the compatible back support to ensure correct and safe installation and use.

To order replacement parts, contact the product provider or Customer Support.


Back support hardware materials: aluminum

Expected product lifetime: 5 years


Compass 4 Hardware - 45 degree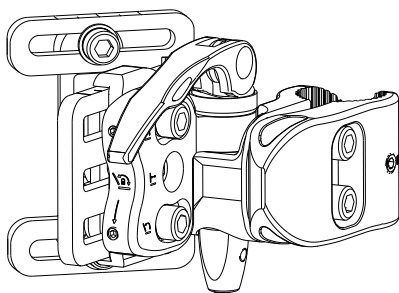


4-Point Hardware (45-degree)

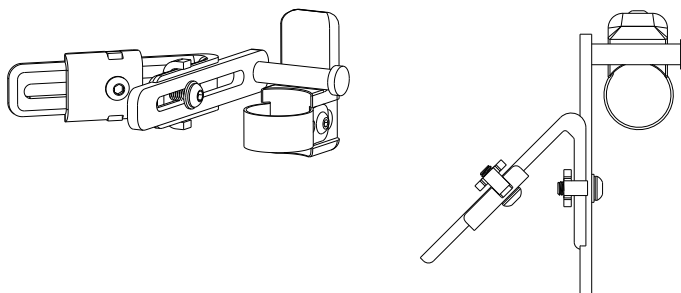
Compass 4 Hardware - 90 degree

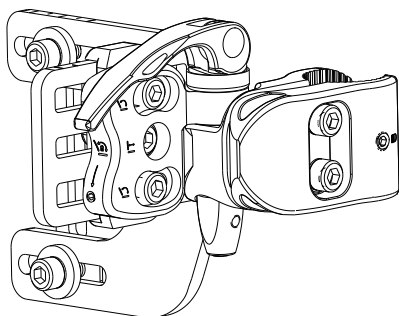4-Point Hardware (90-degree)

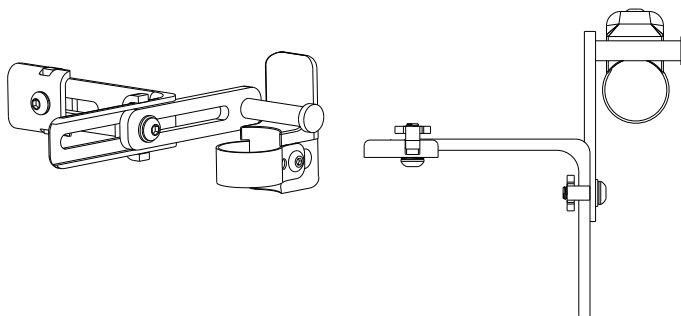⚠️ PSIS WARNING: Make sure that the product is positioned so that the bottom portion makes contact with the Posterior Superior Iliac Spine (PSIS). If the product is NOT positioned to make contact with the PSIS, pelvic and trunk control may be reduced.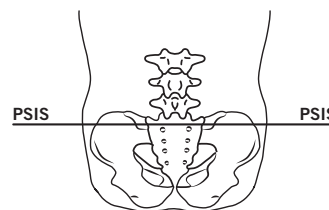

Compass 4 Hardware

Follow these instructions for Compass 4 Hardware and for any back support featured in this manual. For 4-Point Hardware installation, follow the 4-Point Hardware instructions in this manual.

[5mm hex key for bolts; 2mm hex key for locking screws, unless otherwise indicated]

⚠ WARNING: The user **MUST NOT** be seated in the wheelchair during installation or removal of hardware. The user should only transfer to the wheelchair after the hardware has been installed and secured, as instructed.

Install the cane clamps on the wheelchair:



⚠ WARNING: Make sure the wheelchair canes are clean before installing the cane clamps.

⚠ WARNING: If the wheelchair canes have holes, avoid mounting the cane clamps over the holes because the locking screws [J] will not engage with the wheelchair cane surface.

⚠ IMPORTANT! Follow the instructions below for left and right sides.

Recommendation: Mount the Compass 4 Hardware in the middle of the slot on each side of the back shell. Do not mount the hardware in the lower half of the slot.

1. Before installing the cane clamps, use a finger to feel for the locking screws [J] inside the cane clamps. The locking screw should be flush with the inner clamp surface. Adjust the locking screws, if needed.
2. Loosen the bolts [A] until the clamps fit onto the wheelchair canes.
3. Make sure that the clamps are square to the wheelchair.
4. Confirm that the clamps are at the same height (using a ruler or measuring tape).
5. Tighten the bolts [A].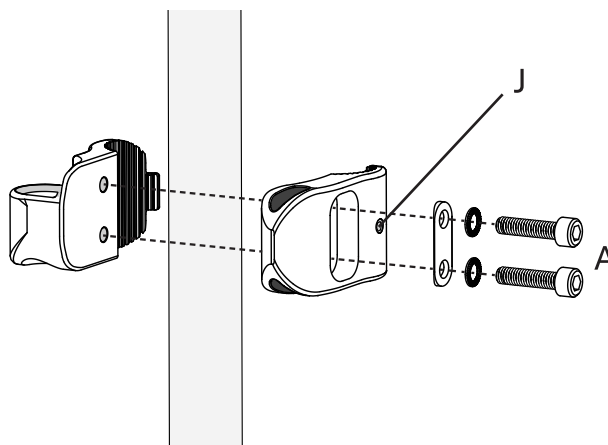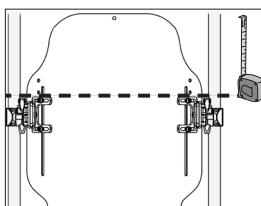

If needed: Attach the hardware to the back shell:

The back shell is provided with the hardware attached. However, if the hardware needs to be reattached, follow these instructions. These instructions apply to both 45-degree or 90-degree brackets.

⚠ WARNING - DROP RISK: The back shell is heavy. Place it on a stable surface while attaching the hardware.


⚠ IMPORTANT! Follow the instructions below for left and right sides.

1. Place the safety washer and then the large washer on each bolt. Place the two bolts/washers through the slot in the back shell and then into the double nut.

Order of assembly:

bolt [B] - safety washer [K] - large washer [L] - angled bracket [M] - back shell - double nut [N]

2. Finger-tighten the bolts, and then slide the hardware toward the desired position. Use the tick marks on the hardware and a measuring tape, if needed, to position the hardware so that it is at the same height and depth on both sides.
3. Tighten the bolts. Repeat on the other side.
4. Follow the instructions on the next page to install the back shell and hardware on the wheelchair.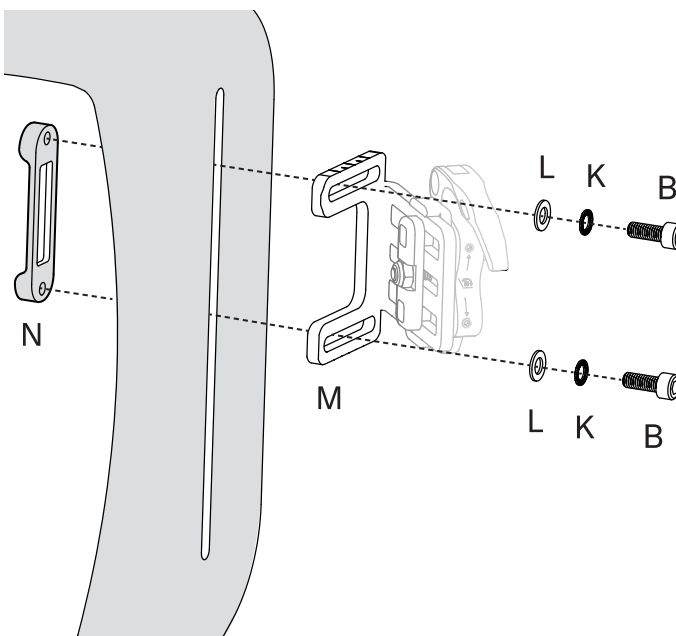

Compass 4 Hardware, continued

Follow these instructions for Compass 4 Hardware and for any back support featured in this manual. For 4-Point Hardware installation, follow the 4-Point Hardware instructions in this manual.

[5mm hex key for bolts; 2mm hex key for locking screws, unless otherwise indicated]

⚠ WARNING: The user may transfer to the wheelchair **ONLY** after the hardware has been installed and secured as instructed. The user **MUST NOT** be seated in the wheelchair during installation.


Install the back shell and hardware on the wheelchair

Recommendation: Mount the Compass 4 Hardware in the middle of the slot on each side of the back shell. Do not mount the hardware in the lower half of the slot.

⚠ IMPORTANT! Follow the instructions below for left and right sides.

1. Slightly loosen the bracket bolts [B] on both sides.
2. Slide the brackets in and out until the wheelchair canes are relaxed. If needed, release the toggle [C] and adjust the clamp angles until the wheelchair canes are relaxed. Lower the toggle to engage the locking mechanism.
3. Securely tighten the bracket bolts [B].
4. Follow the adjustment instructions on the next page.

Shown: Compass 4 Hardware (right side only)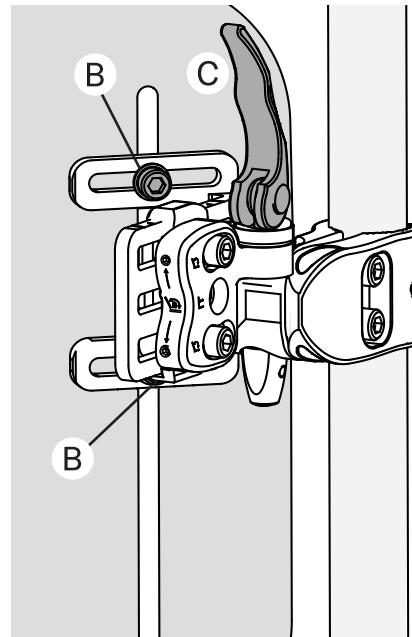

Compass 4 Hardware, continued

Note: Before adjusting, complete installation of Compass 4 Hardware on the wheelchair.

[5mm hex key for bolts; 2mm hex key for locking screws, unless otherwise indicated]

Adjust for height, depth and recline, and secure locking screws:

⚠ WARNING: The user may be seated in the wheelchair during adjustment, as determined safe by a clinician. BEFORE the user transfers to the wheelchair, confirm that the Compass 4 Hardware has been installed according to the installation instructions on the previous page and is securely attached. Make sure the wheelchair wheels are LOCKED.

⚠ IMPORTANT! Follow the instructions below for left and right sides.

⚠ IMPORTANT! When tightening bolts [A] and [G]: In a sequence, repeat the tightening of each bolt until both points are secure and the desired torque is achieved.

1. Height Adjustment (if needed): SLIGHTLY loosen either the clamp bolts [A] or the bracket bolts [B]. Adjust the height, making sure clamps are at the same height on both sides. Securely tighten the bolts. Confirm that the bolts are securely tightened.

2. Depth Adjustment (if needed): Loosen the bolt [F], slide the back shell forward or backward to the desired depth. Securely tighten the bolt on both sides.

Note: The back support can slide forward or backward up to two inches total. If more depth is needed, the hardware can be reconfigured. See the instructions in this manual for alternate configurations.

3. Recline Adjustment (if needed): Loosen the bolts [G], adjust the recline angle, as needed, and then securely tighten the bolts [G].

⚠ WARNING: The locking screws [H] and [J] should be tightened only until contact is made. Over-tightening of the locking screws (H) may cause unwanted changes in the recline position.

4. Lock the Angle Rotation:


On both sides, tighten the locking screws [H] to lock the angle rotation.

Note: The locking screws [H] make contact with the recline bolts [G] to prevent angle rotation. It is possible to also loosen the recline bolts [G], make minor angle adjustments with the locking screws [H], and then tighten the recline bolts [G].



5. Tighten the Cane Clamp Locking Screws:

[3mm hex key]

On both sides, tighten the locking screw [J] to make contact with the wheelchair canes and to lock the cane clamps into position.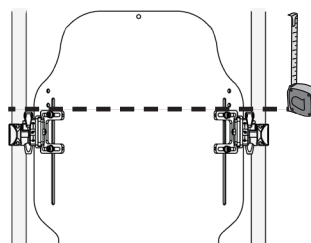

Shown: Compass 4 Hardware (right side only)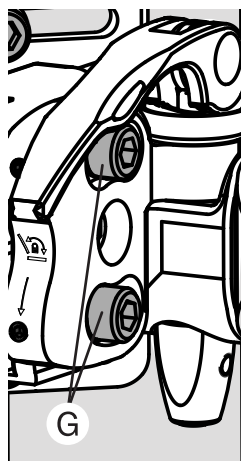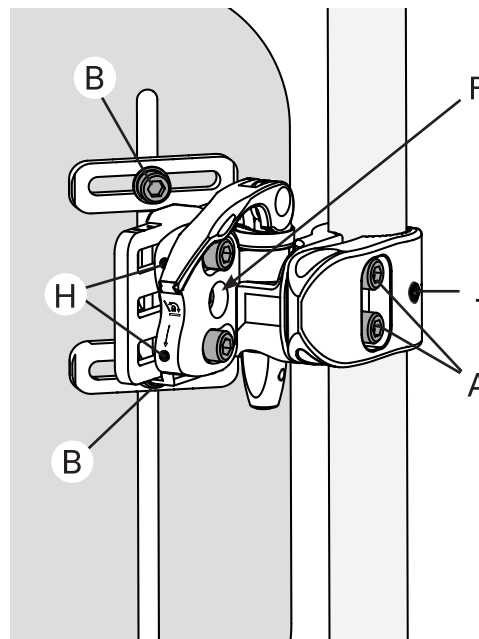

Alternate Configurations


In addition to a standard configuration (as shipped), Compass 4 Hardware can be arranged as described in this section. Each configuration allows for 2 in. (5cm) of depth adjustment.

⚠ WARNING: The user **MUST NOT** be seated in the wheelchair during configuration of the hardware. Follow all hardware instructions and safety information in this manual.


⚠ IMPORTANT! 4-Point Hardware may also require adjustment, if using an alternate configuration of the Compass 4 Hardware. Refer to the 4-Point Hardware instructions in this manual.

⚠ IMPORTANT! To complete alternate configurations, refer to the configuration details below and then follow the Compass 4 Hardware instructions in this manual for **RIGHT** and **LEFT** sides.

45-degree shell bracket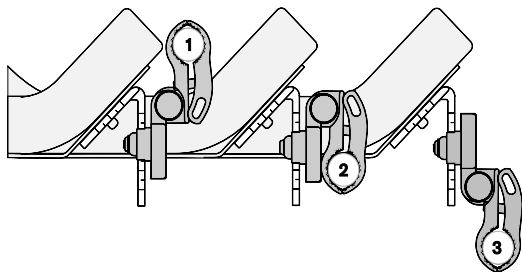


90-degree shell bracket

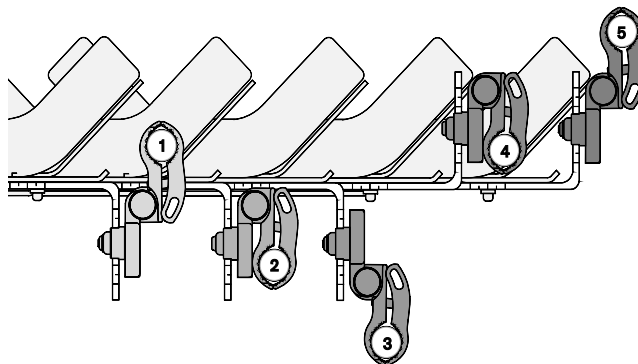Alternate Configuration Options: The “1” shown in the two images directly above represents the standard configuration, which is shown in the Compass 4 Hardware section in this manual.

45 Degree Bracket (compatible with ACTA-BACK and ACTA-Relief)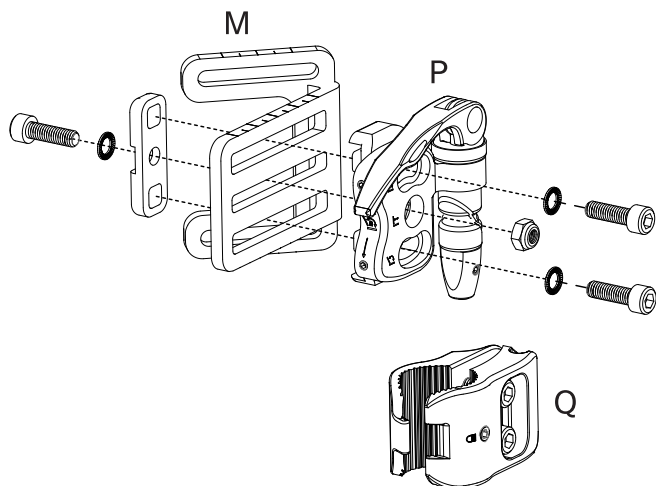


2 Flipped Cane Clamp: RIGHT side shown.

Use the **RIGHT** shell bracket [M].

Use the **RIGHT** toggle [P], but flip the **MIDDLE** screw/washer and bolt.

Use the **LEFT** cane clamp [Q].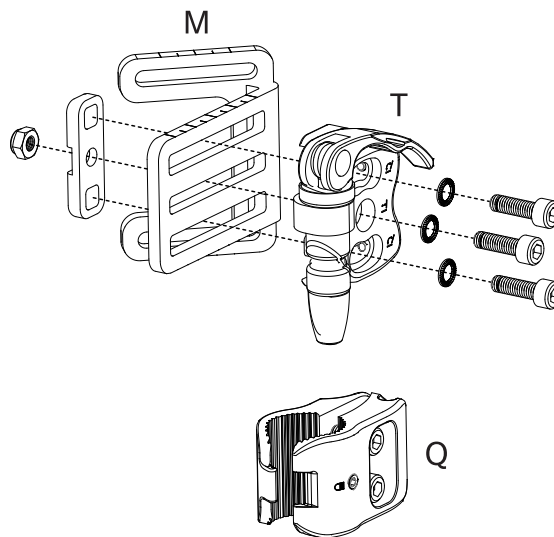

3 Flipped Cane Clamp and Depth Bracket: RIGHT side shown.

Use the **RIGHT** shell bracket [M].

Use the **LEFT** toggle [T].

Use the **LEFT** cane clamp [Q].

Alternate Configurations, continued


90 Degree Bracket (compatible with ACTA-EMBRACE Back)

2 Flipped Cane Clamp RIGHT side shown.

Use the RIGHT shell bracket [M].

Use the RIGHT toggle [P]. Flip the MIDDLE screw/washer and bolt.

Use the LEFT cane clamp [Q].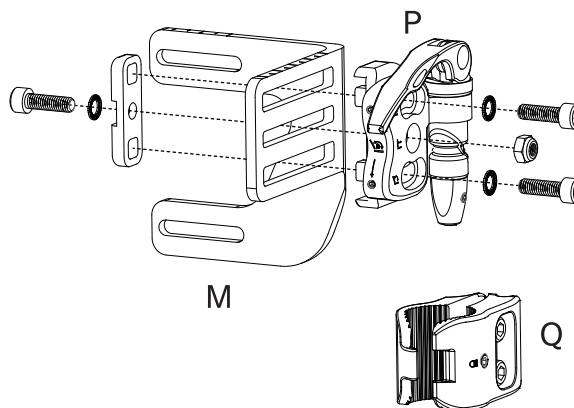


3 Flipped Cane Clamp and Depth Bracket RIGHT side shown.

Use the RIGHT shell bracket [M].

Use the LEFT toggle [T].

Use the LEFT cane clamp [Q].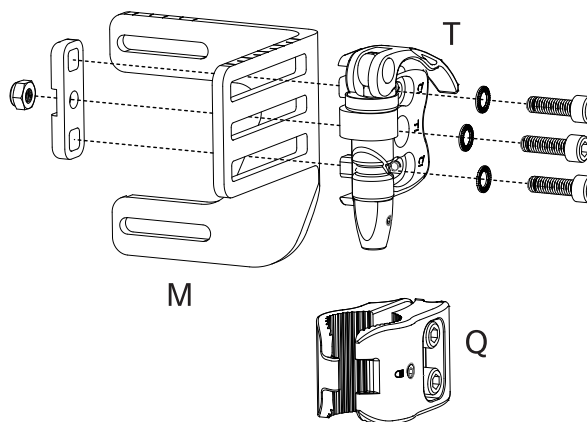


4 Flipped 90° Bracket and Cane Clamp RIGHT side shown.

Use the LEFT shell bracket [U], flipped.

Use the RIGHT toggle [P]. Flip the MIDDLE screw/washer and bolt.

Use the LEFT cane clamp [Q].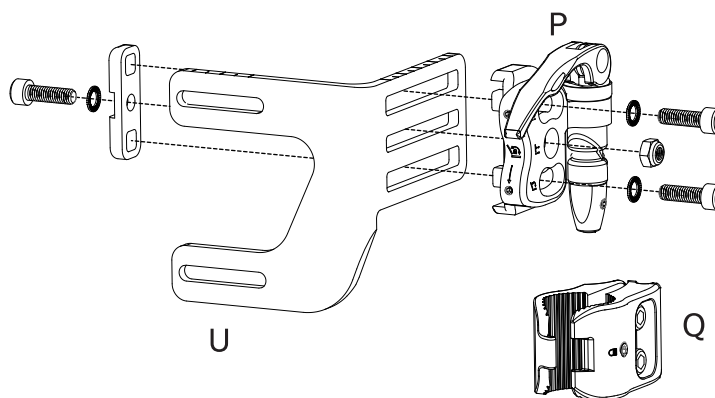


5 Flipped 90° Bracket RIGHT side shown.

Use the LEFT shell bracket [U], flipped.

Use the RIGHT toggle [P].

Use the RIGHT cane clamp [V].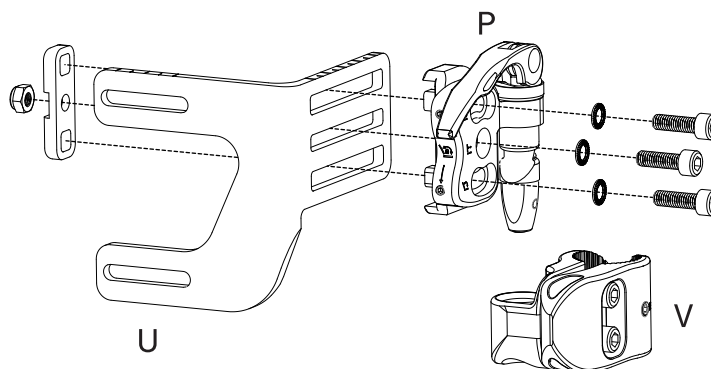

4-Point Hardware

These instructions may be used with 45-Degree or 90-Degree 4-Point Hardware and any back support featured in this manual. 45-Degree hardware is shown.

Before following these instructions, read and follow all instructions for attaching and adjusting the Compass 4 Hardware.

⚠ WARNING: The user may transfer to the wheelchair ONLY after the hardware has been securely installed and tightened as instructed. The user MUST NOT be seated in the wheelchair during installation.

⚠ IMPORTANT! Follow the instructions below for left and right sides.

Attach the angled brackets:


[4mm hex key]

1. Disassemble the brackets: Remove the bolt [A] and safety washer [E].
2. Attach the angled brackets to the back shell in the order shown, and then secure lightly.

Order of assembly:

bolt [A]
safety washer [E]
slot lock bracket [B]
angled bracket [D]
back shell
slot lock [C]

3. Slightly loosen the bolts [A] and [K].
4. Adjust the hardware so that the flat part (not the end) of the mounting peg rests against the wheelchair cane.
5. Securely tighten the bolts [A] and [K].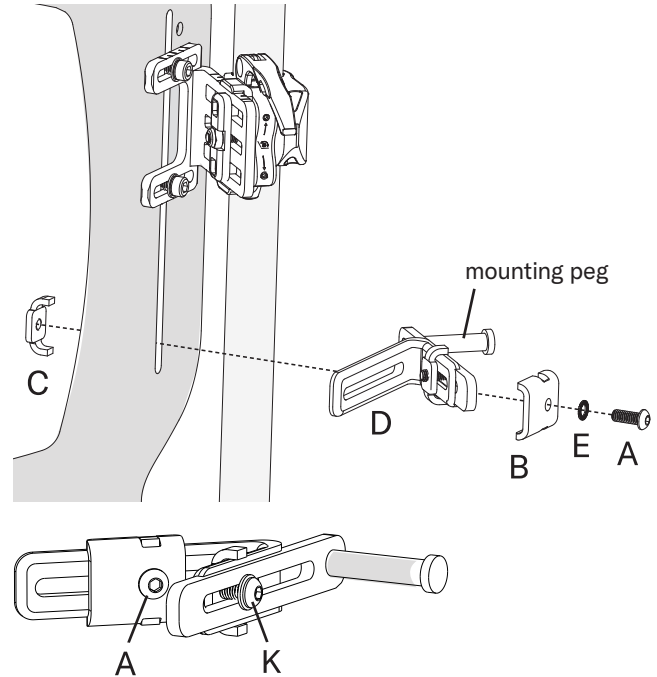


Attach the cane mounts:

[2mm hex key]

6. Release one side of the cane strap by removing the screw [F] and safety washer [J].
7. Select/adjust the cane straps: Place the cane strap around the wheelchair cane to see how it fits. Depending on the size of the wheelchair canes, both sides of the cane strap may need to be removed to switch to a different attachment hole on one or both sides, or the larger cane strap may be needed.
8. Install the cane mount on both sides: Place the strap around the cane. Attach the other side of the cane strap with the screw [F] and safety washer [J] so that the fit is as snug as possible and tighten.
9. To secure the straps, tighten the screw [M].

After installation is complete, the user may transfer to the wheelchair.

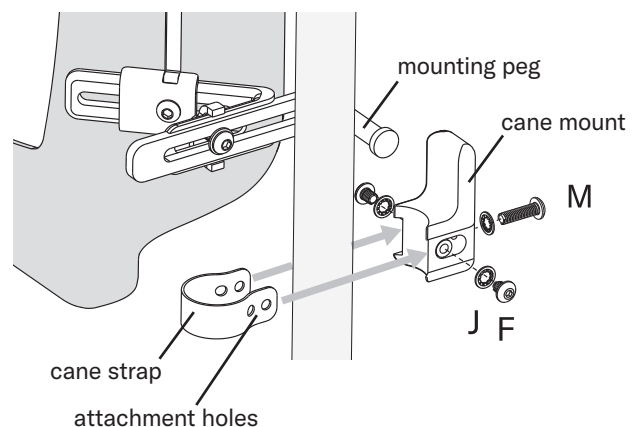Troubleshooting

For additional assistance, contact the equipment provider, distributor, or Customer Support.

Cane clamps slide on wheelchair canes.	<ul style="list-style-type: none"> - Confirm that the wheelchair cane diameter is: ¾ in. (19mm), ⅞ in. (22mm), 1 in. (25mm) or 1⅛ in. (29mm). If it is not, the wheelchair canes are not compatible with the Compass 4 Hardware. - Make sure the bolts and locking screw on each cane clamp are securely tightened.
Not able to mount the cane clamp onto the cane - the clamps are too far from the canes.	<ul style="list-style-type: none"> - With the toggle lifted, rotate the clamps in or out. - Loosen the bolts and slide the brackets that mount to the back shell.
Not able to access the attachment bolts when installing hardware on the back shell.	<p>To access the attachment bolts, first adjust the attachment hardware:</p> <ul style="list-style-type: none"> - Depth bolts: Loosen the depth adjustment bolt and move to the outermost position; then loosen the center bolt and slide into position. - Recline bolts: Make sure the lock screws are loosened, and then loosen the bolts, adjust the recline, and re-tighten the bolts.
Cane clamps / hardware continue to rotate after installation is complete.	Make sure all locking screws have been tightened, as instructed in this manual.
The 4-Point Hardware cane straps do not reach around the wheelchair canes.	For some wheelchair canes, the longer cane strap (included with the 4-Point Hardware) may be needed. See the instructions for 4-Point Hardware.
Compass 4 Hardware: The toggle is either too tight to lower into position or too loose and will not stay in the lowered position.	Contact Customer Support for assistance.
A component is damaged or missing.	Contact Customer Support for a replacement.

Cleaning and Disinfecting

CAUTION: Cleaning and disinfecting are separate processes. Cleaning must precede disinfection. Before use by a different individual: clean, disinfect, and check product for proper functioning. Clean the product regularly and when it becomes soiled.

Recommendation: When washing by hand, use room-temperature water.

Cleaning: Hand wash with a sponge, using soap and water. Rinse with clean water. Wipe dry with a clean cloth and then allow to air dry completely.

Disinfecting: Wipe gently with a cloth dampened with a household disinfectant. Rinse with clean water. Wipe dry with a clean cloth and then allow to air dry completely.

Care Symbols

Hand wash.

Dry flat.

Wipe with a damp cloth.

Maintenance

⚠ WARNING: DO NOT continue to use if any part cannot be tightened or is broken. Periodically, the product should be thoroughly inspected by the equipment provider.

Frequently: confirm the tightness of attaching hardware; re-tighten loose components, following the instructions in this manual.

Storage, Transport, Disposal and Recycling

Storage or Transport: Clean and disinfect the product. Place in a container that protects the product from moisture, contaminants and damage. After storing or transporting, inspect the product to ensure it was not damaged (see "Troubleshooting"), and follow instructions to prepare the product for use.

Disposal: The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of its useful life, treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incineration must be performed by a qualified, licensed waste management facility.

Recycling: Contact a local recycling agency to determine recycling options for the product.

Warranty

This product is covered by a lifetime warranty against manufacturer defects. If a manufacturing defect should occur, discontinue use immediately; note that the product will be replaced at no cost to the buyer. Wear from regular use is not considered a manufacturing defect. Replacement parts are available by contacting your supplier, distributor, or retailer.

The following will void the warranty: modifications made in attempt to interface with other products (where compatibility is limited, as described in product information); alterations made to the product; or defects caused by irregular use.

Warranty Replacement Procedure

Warranty claims should be initiated by the original purchaser, who should contact the DME supplier, distributor or retailer from whom the product was purchased, if they are available, or any authorized Permobil DME supplier, distributor or retailer.

Whenever possible, the DME supplier, distributor or retailer with a warranty issue should contact Customer Support and provide the original purchase order number, sales order number, or invoice number.

Before a warranty replacement order or quote may be processed, Customer Support must issue a Return Merchandise Authorization (RMA).

Información de contacto

 The Comfort Companies, LLC
1960 South Calhoun Road
New Berlin, Wisconsin 53151 EE. UU.

Teléfono:
EE. UU.: +1 800-736-0925
Fuera de EE. UU.: +1 618-277-9150
Fax: +1 406-522-8563

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Permobil LTD