

HOW TO ASSEMBLE YOUR



OVERBED TABLE TILT-TOP

Item # 6071


Capacity not to exceed 22 lbs



- a. (1) Table top
- b. (1) Column
- c. H Base w/ Wheels

PARTS INCLUDED

- d. (1) Knob & washer
- e. (1) Triangular knob
- f. (2) Mounting screws & nut
- g. (1) Allen & Wrench

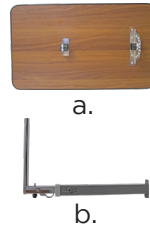

a.



c.



e.

f.



g.

Follow these SIMPLE STEPS



Quick & Easy Assembly

1 Locate the 2 holes on the H base and align with the 2 column holes. Secure the column to the H base with the 2 mounting screws and nut with the allen & wrench (see figure 1).

Note: Ensure that the column bar is facing the correct way as (figure 1).

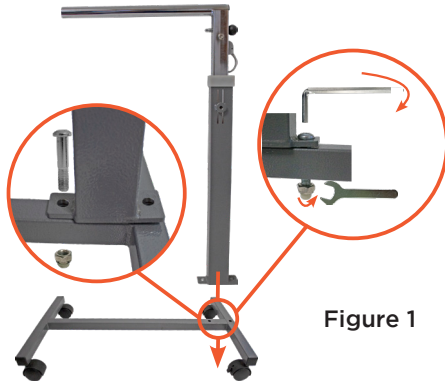


Figure 1

2 Slide the table top all the way into the column and align the center adjustments holes to the pin on the column (see figure 2).



Figure 2

3 Secure the table top by inserting the washer and knob at the end of the column and fully tighten the knob (see figure 3).

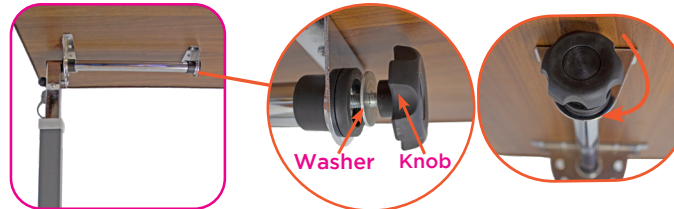


Figure 3

4 Pull the lever located below the pin and pull up from the bottom of the table top until you reach your desired height (see figure 4).

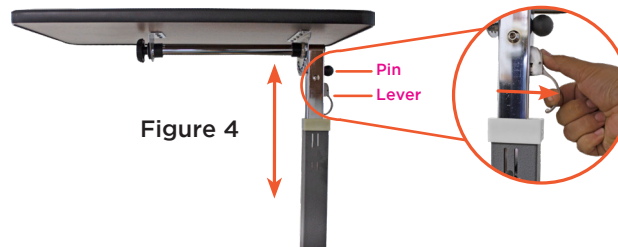


Figure 4

5 Attach the triangular knob on the lower column bar (see figure 5). **Note:** Do not tighten the knob until the desired height is reached on figure 6.

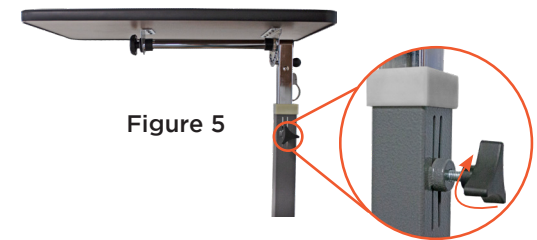


Figure 5

6 Pull pin out and adjust to desired angle and release pin through the adjustment holes (see figure 6). Tighten the triangular knob from figure 5.

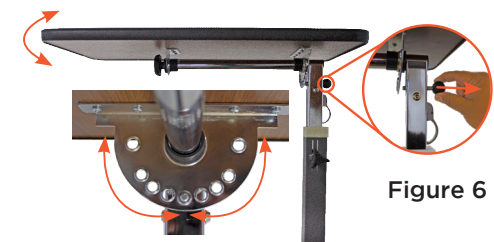


Figure 6

7 **To lock the wheel:** Step down on the wheel locking lever figure 7.

To unlock the wheel: Pull up on the wheel locking lever figure 7.



Figure 7

*Revised June 9, 2021

WARNING

Read all instructions carefully and consult with a qualified professional before assembling to avoid possible injury. **NOVA Medical Products shall have no liability for any injury resulting from improper assembly or misuse of this product.**

- **DO NOT** over tighten the bolts, screws or use any powertolls when assembling this product.
- **DO NOT** use the product if it appears to be damaged or unstable.
- **DO NOT** modify the product in any way.
- **DO NOT** Overload the product as that can cause it to brake and/or serious injury.
- Check frequently for cracks, damage, or looseness.
- Be careful not to pinch your fingers when attempting to tilt the table top.
- Maximum weight capacity is 22lbs.

CARE AND MAINTENANCE

Check to make sure all hardware is securely tightened before using the table. Use a mild detergent and warm water to clean. Dry with a soft cloth to avoid moisture build-up.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts	One Year Limited Warranty (rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.


THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US!
We're here to help


 info@novajoy.com


 1 800 557 6682


novajoy.com


Connect with us!

novajoy.com

 [novamedicalprod](https://www.facebook.com/novamedicalprod)

 [novamedicalprod](https://twitter.com/novamedicalprod)

 [novawalkerstalk](https://twitter.com/novawalkerstalk)

 [NOVA Medical Products](https://www.youtube.com/channel/UC...)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE